


# SARS-CoV-2 CE

## Antigen Rapid Test Kit

(Colloidal Gold Immunochromatography)

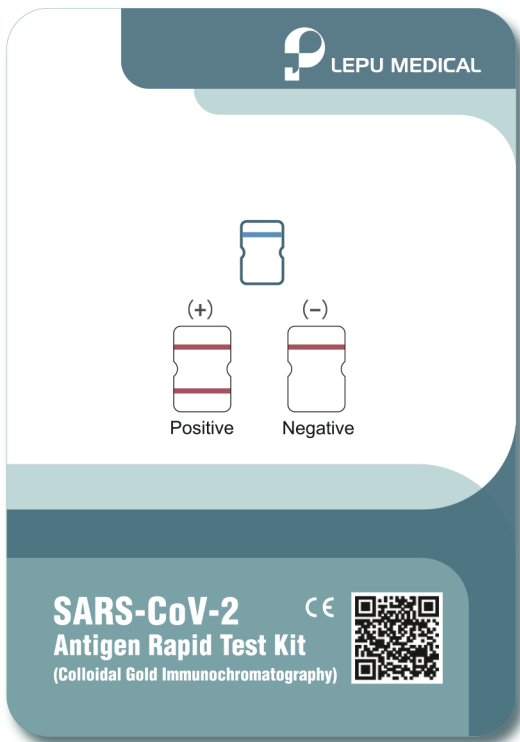

- ① Non-invasive
- ② Simple to use
- ③ Convenient, no devices required
- ④ Rapid, get result in 15 minutes
- ⑤ Stable, with high accuracy
- ⑥ Inexpensive, cost-efficiency



Hospital



Test Site



Airport



Station



Hotel




Corporation




Mass  
Screening




## Instruction

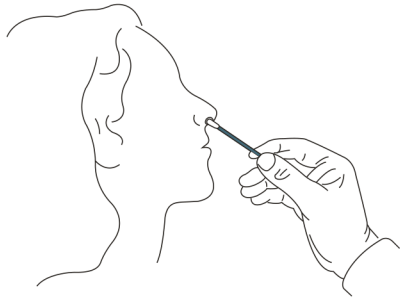


Step 1: Use swab to collect sample.

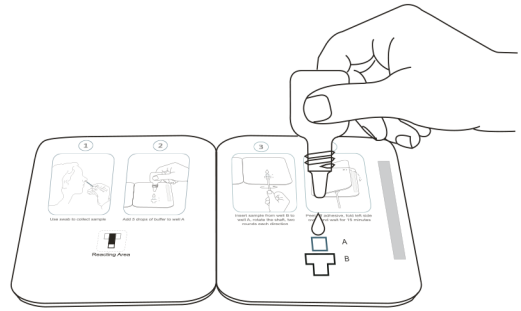

Step 2: Add 5 drops of buffer to well A.

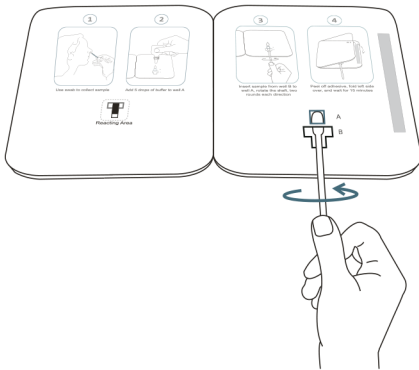

Step 3: Insert the sample through well B to well A, rotate the shaft clockwise first and counterclockwise, twice each.

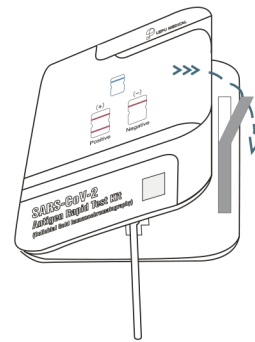

Step 4: Peel off the adhesive, fold the left side over, and wait for 15 minutes.



## Result Interpretation


Positive

(+)



Negative

(-)

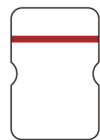

Invalid

